

Clinician's Quick Set-Up Guide for the NIPPY Clearway

About the Device

- The NIPPY Clearway is a mechanical insufflation-exsufflation device (MI-E).
- The NIPPY Clearway is licensed as a therapy device and can be used for adults and paediatric patients as directed by a competent physician.
- The NIPPY Clearway can be used with a face mask, via a mouthpiece or connected directly to the tracheostomy tube.

Unlocking and Locking the Device

- To unlock the device, push and hold simultaneously the + and – button for 3 to 4 seconds until the padlock on the bottom right of the screen unlocks.
- To lock the device, again, hold the + and – button simultaneously for 3 to 4 seconds until the padlock on the screen shows as locked.

Clear Settings

- To clear the settings, press the menu button and then select option 4 – clear patient settings. Then follow the instructions on the device.

Setting Up Manual Mode

- Once the settings are cleared you will be in manual mode as default.
- To set the insufflation pressure, press the insufflation button, then use the + and – keys to select the required pressure, confirm by pressing set.
- To set the exsufflation pressure, press the exsufflation button, then use + and – keys to select the required pressure, confirm by pressing set.
- Using either the manual control or front button, you can deliver the therapy manually. You control the insufflation and exsufflation times as well as the pause time manually.

Setting Up Basic Auto Mode

- To select basic auto mode, press the mode button, use the – key to scroll down to basic auto mode, number 2 in the menu, and press set to select.
- To set the insufflation and exsufflation pressures, press the insufflation or exsufflation button, then use the + and – keys to select the required pressure, confirm by pressing set.
- To set the insufflation (Ti) and exsufflation times (Te), select the appropriate button, Ti or Te and use the + or – keys to set the time, confirm the correct time by pressing set.
- To set the pause, press the pause button, then using the + or – keys select the appropriate pause time, confirm this by pressing set.
- To start therapy, press the start button, this will give maximum 20 cycles, to stop the therapy before this, press the start button again.

Consumables

NIPPY Clearway Circuit		0960.101
Full Face Mask		0960.106 /? I = Infant P = Paediatric S = Small M = Medium L = Large XL = Extra Large
Mouthpiece & Nasal Clip		0960.105
Flexible Catheter Mount		0960.102
Anti-Bacterial Filter		0960.107
Oxygen Coupler		0960.108
Air Inlet Filter		0960.116
Trolley		0960.115
External Battery		0960.111
External Battery Charger		0960.112
Manual Control		0960.109
Mobility Bag		0960.114